

# C1 Daily Function Checks

---

The function check comprises a visual inspection of the outer housing and a function check of the functionalities and options of the **C1**.

Additionally, inspect the test box and cable / test load for visible damage prior to use.

**Do not use** the test box / test load if visibly damaged.

---

## Powering On

 Press the **On / Off** key to power the device on

 *An Internal test will be automatically complete*

---

## Defibrillator / Pacer

-  Connect the Testload to the therapy socket
- ⇒ Press the [Yes] softkey to confirm the message  
**"Mark mission as test mission?"**
  - ⇒ Select Manual mode  
Select an energy level of 50 J
  - ⇒ Charge
  - ⇒ Deliver the shock

 *Check the shock protocol printout for the message  
"Test successful" OR "Test failed"*

*The energy is internally discharged via the Testload*

---

## Power Supply

 Check state of charge of the **C1** when powered by the batteries only

 Connect to mains supply

 Check state of charge of the compact device, observed in %

 *State of charge of the battery is greater than 30%*

---

## Accessory Checks

  All patient monitoring and therapy intermediate cables are present, plugged into the device and undamaged.

  There is an appropriate supply of all patient monitoring and therapy accessories.

  All re-usable accessories are undamaged.

  Packaging for single patient use consumables is intact, undamaged and within the expiry date.

---

## Powering Off

 Press the **On / Off** key to power off the device