ACCUVAC Pro/ACCUVAC Lite

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
ACCUVAC Rescue/ACCUVAC Basic
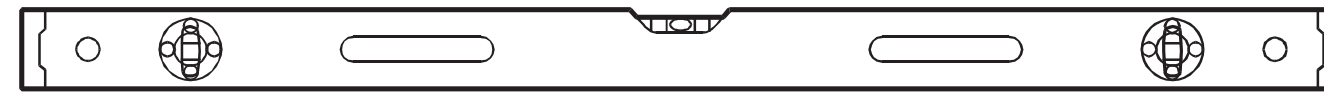


General information

- The outline of the ACCUVAC on this drilling template represents the space required by the device and its components.
- When fitting, take account of the specific route of removal and the space required for the hands when removing it.
- The fixing surface must be sufficiently strong to reliably carry the wall mounting and the ACCUVAC to EN 1789 Ambulances.
- Only use the sheet-metal screws enclosed on metal mounting surfaces, e.g. in vehicles. For non-metal mounting surfaces, use screws to suit the surface in question (not included in scope of supply).

Fitting the wall mounting

1. Select a level surface and vertical mounting location.
2. Hold up the drilling template and align it using a spirit level.
3. Mark the two left-hand holes at the mounting location.
4. Remove the drilling template from the mounting location.
5. Hold the wall mounting up to the mounting location.
6. Mark the two right-hand holes with the aid of the wall mounting.
7. Drill the holes for mounting using a $\varnothing 3$ mm drill bit.
8. Screw the wall mounting tight using the enclosed sheet-metal screws.
9. Push the connector of the 12 V connecting cable (all ACCUVAC models) or of the plug-in charger (ACCUVAC Basic only) into the guide rail of the wall mounting until it engages.

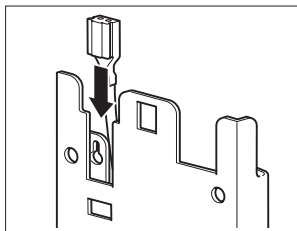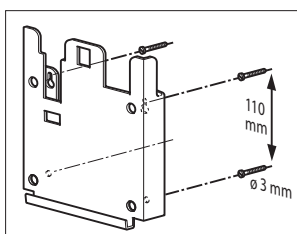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