

## Warrior lite Blood and Fluid Warmer *Mount Adapter* Installation Guide

- 1. Purpose:** Detail the *installation instructions* of the *Warrior lite Mount Adapter* (P/N: QIF03-MUA1000) and describe its *cleaning instructions*.
- 2. Product Description:** The *Warrior lite Mount Adapter* is connected to the back side of the *Warrior lite Base Unit*.

The *Mount Adapter* can be connected to off the shelf versatile *Mount clamps* (such as GCX P/N PRC-0001-01\*, see figure 1b), see figure 2;



**Figure 2: Versatile Mount Clamp attachment to Pole or Rail**

Strap\*, using the slits (see figure 1a) , see figure 3, or hook\* (see figure 1d)




**Figure 3: Warrior lite mounted to a Pole using a strap**


\* The *Mount accessories* are not provided with *Warrior lite* system

### **3. Installation Instructions to the Base Unit**


- Locate the *Mount Adapter* on the back side of the *Base Unit*, with the fixation snap located as marked in figure 4a and 5.

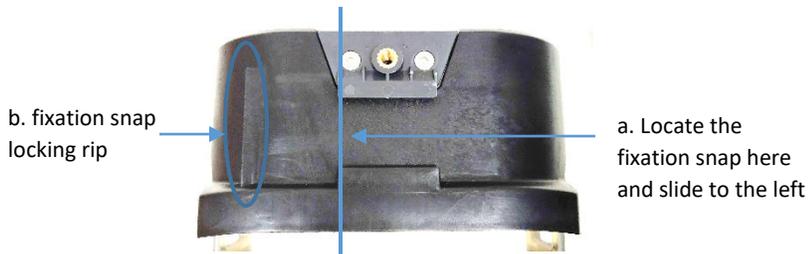

**Figure 4: Warrior lite Base Unit back side**

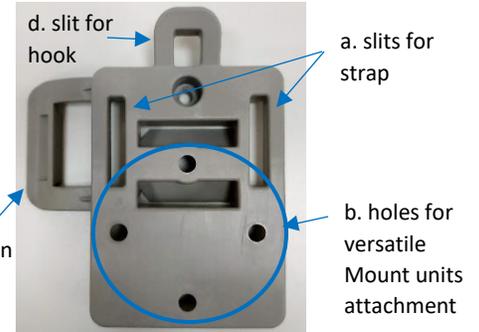

**Figure 1: Mount Adapter front**

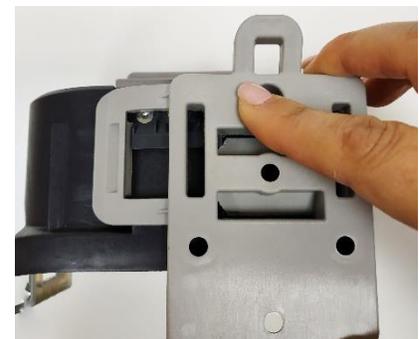

**Figure 5: Locating the *Mount Adapter* on the *Warrior lite Base Unit* back side, then sliding it to the left**

b. Slide the *Mount Adapter* to the left, until the fixation snap locks on the fixation snap locking rib (see figure 4b, see figure 6)



**Figure 6: *Mount Adapter* attached to the *Warrior lite Base Unit* back side**



**Figure 7: *Mount Adapter* attached and fixed with a screw to the *Warrior lite Base Unit* back side**

In case the *Warrior lite* is intended to remain fixed on the pole / rail, and an extra fixation is required, there is an option to fix the *Mount Adapter* to the Base Unit using an M4x8 screw, as seen in figure 7.

C. Routine maintenance: periodically inspect all hardware associated with the *Mount Adapter*. Tighten or adjust as necessary for optimal operation and safety.

#### 4. Cleaning Instructions:

- The *Warrior lite Mount Adapter* is a reusable device.
- Clean the *Warrior lite Mount Adapter* after each use.
- Cleaning should consist of wiping the external covering of the *Warrior lite Mount Adapter* with a damp cloth with Deconex® SOLARSEPT solution (by Borer Chemie AG).
- Do not submerge, sterilize or autoclave the *Warrior lite Mount Adapter*.