



Warrior lite Blood & Fluid Warmer Skill Sheet

Provider Name: _____ Organization: _____

Date: _____ Instructor: _____

The following evaluation form is in accordance with the recommended steps per IFU and best practice ^[1]:

Steps	Pass/Fail	Comments
Preparation		
Places the battery properly		
Secures the battery using the side latches		
Ensures the CDU is not expired		
Connection		
Prepares appropriate tubing of choice ^[2]		
Spikes the fluid of choice ^[3]		
Connects the tubing to CDU using “IN” luer lock		
Operation		
Primes the tubing from all air		
Places the CDU on the BU (or connects it to an Extension Cable) ^[4]		
Connects the “OUT” to the patient		

Notes:

- [1] This evaluation form is for training purpose only; it does not replace the IFU
- [2] Tubing of choice: The Warrior lite does not require a proprietary tubing
- [3] Fluid of choice: any clear fluid or blood product (excluding standalone platelets – USA customers only)
- [4] Ensure tubing and CDU are primed prior to connecting it to the Warrior lite

Lexicon:

- BU:** Base Unit
- CDU:** Compact Disposable Unit
- IFU:** Instructions For Use